All4Labels
GLOBAL PACKAGING GROUP

AI4Labels Hamburg GmbH & Co. KG
Möllner Landstr. 15
D-22969 Witzhave
Phone +49 4104 693 0
Fax +49 4104 693 110
Mail info@all4labels.com
Web www.all4labels.com

Stanza

P Cool Gray 10 C

1

2

3

4

5

6

7

8

9

10

Product Name	Mat.No.	Index	Dimensions	Package	Country	EAN
Vaginal Applicator	20005293	A	210 x 297 mm	BPZ	EN, DE, FR, IT, GR, HR, HU, PL, RS, AR	20005293-A
Application Area / Category	Label No.	Version	Techn. Spec.	Date	Operator	Font Size
Antisepsis / MP	3050	03	Auftragsherstellung	19.08.2024	JG	6 pt - 95%

Fertigwarennummer: 70001128
alt: 20004071-A / 305036620_00
Techn. Spec. 133

Exclusion of liability for the implementation of statutory provisions and color liability. This PDF is not color-coded, it only serves to check text, status and color distribution. Compliance with applicable statutory provisions with regard to font sizes, industry-sector-specific labeling requirements, trademark, competition and licensing laws must be checked and ensured by the customer.

Please use the overprint option for PDF preview.

schülke -+

VAGINAL APPLICATOR

EN

for use in combination with octenisept® solution

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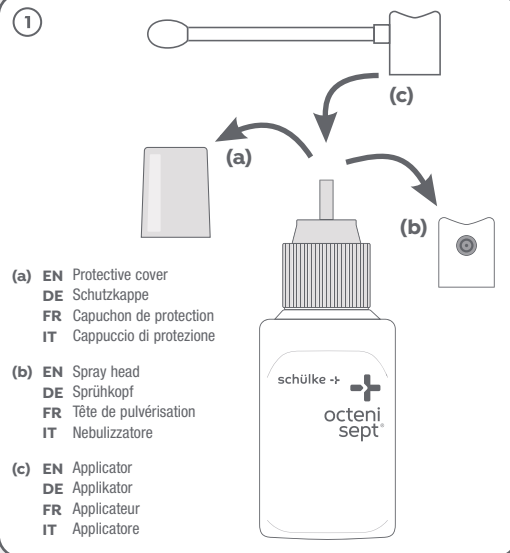
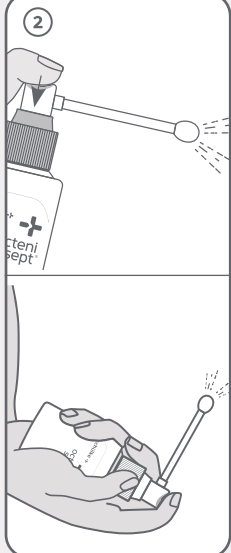
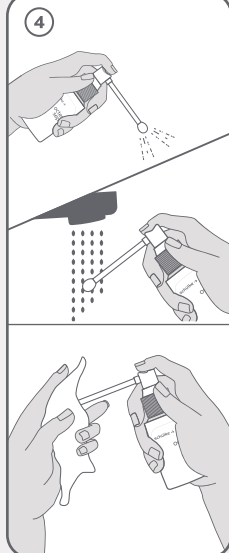
4

EN Information for use

- Prior to first use, take this applicator (c) out of the wrapper and, after removing the transparent protective cap (a) and spray head (b), place it on the octenisept® bottle.
- Prior to first use, operate the spray dispenser eight times to ensure the correct dosage.
- We recommend that you insert the applicator deeply into the vagina while sitting on the toilet. Now, press the applicator's cap ten times with your index finger to distribute the preparation deep within the vagina.
- Hold the applicator downwards and spray some of the product out, for example, over the sink. Afterwards, rinse off the applicator under running water and dry it with a clean cloth.
- For each subsequent application, simply reattach the applicator and press the spray head until some of the preparation is released.

The applicator can be used for the duration of a treatment cycle. After the end of the vaginal therapy, the applicator should be discarded in household waste. The applicator is not suitable for patients under the age of 8. The applicator is to be used exclusively for the intravaginal application of octenisept® and it is not suitable for other body orifices (such as the nose, ear, anus).

Please read instructions for use of octenisept®. Any serious incidents related to the product are to be reported to the manufacturer and competent authority.

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